



CROW (Charcot Restraint Orthotic Walker)

General Description:

CROW is an acronym for Charcot Restraint Orthotic Walker. This orthosis is prescribed for patients who have foot ulcers or insensate feet (can't feel). This is an orthosis that is clamshell in design and covers the entire foot and calf of the leg, resembling a ski boot (See Figure 1). While it is somewhat big and bulky, the CROW gives tremendous support by preventing foot and ankle movement. It is fully padded on the inside. A shoe is not worn with this orthosis.



Bannockburn 2101 Waukegan Road, Suite 104 Bannockburn, IL 60015 847-444-0690	Lincoln Park/Chicago 2551 N. Clark St., Suite 200 Chicago, IL 60614 773-472-3663	Rush/Chicago 1725 W. Harrison St., Suite 220 Chicago, IL 60612 312-942-2011	UIC/Chicago 1740 W. Taylor St. Room C100 Chicago, IL 60612 312-996-6450	Hickory Hills 8641 W. 95 th St. Hickory Hills, IL 60457 708-599-8336
Naperville 1551 Bond St., Suite 111 Naperville, IL 60563 630-637-4638	Oakbrook Terrace 1S376 Summit Ave., Ct. E Oakbrook Terrace, IL 60181 630-424-0392	Oak Park 401 Harrison Street Oak Park, IL 60304 708-383-2257	Schaumburg 1701 E. Woodfield Rd., Ste 555 Schaumburg, IL 60173 847-619-1701	Schererville 6629 W. Lincoln Hwy. (Rt. 30), Suite 1 PO Box 507 Schererville, IN 46375 219-864-9501



Application:

1. Apply a cotton stockinette or a long, thin, cotton sock to leg (See Figure 2).
2. Slide the CROW into position, ensuring that the heel is fully seated within the CROW (See Figure 3). The heel needs to be back all the way and in contact with the bottom of the footplate. Check by looking along the sides and the back where it is separated above the heel.
3. Apply the front section of the device, making sure it's sides overlap the back section (See Figure 4).
4. Apply the instep strap on the front of the ankle (See Figure 5).
5. Fasten all remaining straps (See Figures 6 and 7).
6. Skin should be checked for redness that does not go away after approximately 15 minutes. Slight redness is common over the instep and under the ball of the foot.



Figure 2



Figure 3

Bannockburn
2101 Waukegan Road,
Suite 104
Bannockburn, IL 60015
847-444-0690

Naperville
1551 Bond St., Suite 111
Naperville, IL 60563
630-637-4638

Lincoln Park/Chicago
2551 N. Clark St., Suite 200
Chicago, IL 60614
773-472-3663

Oakbrook Terrace
1S376 Summit Ave., Ct. E
Oakbrook Terrace, IL 60181
630-424-0392

Rush/Chicago
1725 W. Harrison St.,
Suite 220
Chicago, IL 60612
312-942-2011

Oak Park
401 Harrison Street
Oak Park, IL 60304
708-383-2257

UIC/Chicago
1740 W. Taylor St.
Room C100
Chicago, IL 60612
312-996-6450

Schaumburg
1701 E. Woodfield Rd.,
Ste 555
Schaumburg, IL 60173
847-619-1701

Hickory Hills
8641 W. 95th St.
Hickory Hills, IL 60457
708-599-8336

Schererville
6629 W. Lincoln Hwy.
(Rt. 30), Suite 1
PO Box 507
Schererville, IN 46375
219-864-9501



SCHECK & SIRESS

PROSTHETICS • ORTHOTICS • PEDORTHICS



Figure 4



Figure 5



Figure 6



Figure 7

Bannockburn

2101 Waukegan Road,
Suite 104
Bannockburn, IL 60015
847-444-0690

Naperville

1551 Bond St., Suite 111
Naperville, IL 60563
630-637-4638

Lincoln Park/Chicago

2551 N. Clark St., Suite 200
Chicago, IL 60614
773-472-3663

Oakbrook Terrace

1S376 Summit Ave., Ct. E
Oakbrook Terrace, IL 60181
630-424-0392

Rush/Chicago

1725 W. Harrison St.,
Suite 220
Chicago, IL 60612
312-942-2011

Oak Park

401 Harrison Street
Oak Park, IL 60304
708-383-2257

UIC/Chicago

1740 W. Taylor St.
Room C100
Chicago, IL 60612
312-996-6450

Schaumburg

1701 E. Woodfield Rd.,
Ste 555
Schaumburg, IL 60173
847-619-1701

Hickory Hills

8641 W. 95th St.
Hickory Hills, IL 60457
708-599-8336

Schererville

6629 W. Lincoln Hwy.
(Rt. 30), Suite 1
PO Box 507
Schererville, IN 46375
219-864-9501

Wearing Schedule:

On the first day you receive the brace, begin by wearing for only 1 hour. After 1 hour, remove the brace and check your skin for red marks. Some small, light red marks may be noticed on the skin that should go away in 20 to 30 minutes after removing the brace. Slight redness is common over the instep and under the ball of the foot. If the red marks do not go away in 20 to 30 minutes or if you notice any scratching, bruising, or blistering, **do not put the brace back on**. Call immediately to schedule an appointment with your orthotist.

If the skin is ok, wait at least 1 hour and then put the brace back on for 1 hour at a time for the rest of the first day, checking the skin after each hour.

On the second day, put the brace on for 2 hours. After 2 hours, remove the brace and check the skin. If the skin is ok, put the brace back on for 2 hours at a time for the rest of the day, checking the skin after every 2 hours. If your skin is ok, gradually increase wearing time by 1 hour each day, checking the skin after each wearing time.

Cleaning and Maintenance:

The best way to clean a CROW is to spray the inside with rubbing alcohol and wipe dry to remove body oils and residue. It can also be cleaned by wiping it out with a damp towel and anti-bacterial soap, or anti-bacterial moist towelettes.

Do not immerse the CROW in water, as this will harm the straps and fasteners. Keep the CROW away from excessive heat to prevent damage to the plastic.

Tips and Problem Solving:

- Proper footwear should be worn at all times on the opposite foot.
- If you notice red marks, bruises, or blisters on your skin, discontinue wearing the brace and call your orthotist to schedule an appointment.
- If there are any problems or questions, please contact the orthotist.

<p>Bannockburn 2101 Waukegan Road, Suite 104 Bannockburn, IL 60015 847-444-0690</p> <p>Naperville 1551 Bond St., Suite 111 Naperville, IL 60563 630-637-4638</p>	<p>Lincoln Park/Chicago 2551 N. Clark St., Suite 200 Chicago, IL 60614 773-472-3663</p> <p>Oakbrook Terrace 1S376 Summit Ave., Ct. E Oakbrook Terrace, IL 60181 630-424-0392</p>	<p>Rush/Chicago 1725 W. Harrison St., Suite 220 Chicago, IL 60612 312-942-2011</p> <p>Oak Park 401 Harrison Street Oak Park, IL 60304 708-383-2257</p>	<p>UIC/Chicago 1740 W. Taylor St. Room C100 Chicago, IL 60612 312-996-6450</p> <p>Schaumburg 1701 E. Woodfield Rd., Ste 555 Schaumburg, IL 60173 847-619-1701</p>	<p>Hickory Hills 8641 W. 95th St. Hickory Hills, IL 60457 708-599-8336</p> <p>Schererville 6629 W. Lincoln Hwy. (Rt. 30), Suite 1 PO Box 507 Schererville, IN 46375 219-864-9501</p>
--	--	--	---	---



SCHECK & SIRESS

PROSTHETICS • ORTHOTICS • PEDORTHICS



Bannockburn

2101 Waukegan Road,
Suite 104
Bannockburn, IL 60015
847-444-0690

Naperville

1551 Bond St., Suite 111
Naperville, IL 60563
630-637-4638

Lincoln Park/Chicago

2551 N. Clark St., Suite 200
Chicago, IL 60614
773-472-3663

Oakbrook Terrace

1S376 Summit Ave., Ct. E
Oakbrook Terrace, IL 60181
630-424-0392

Rush/Chicago

1725 W. Harrison St.,
Suite 220
Chicago, IL 60612
312-942-2011

Oak Park

401 Harrison Street
Oak Park, IL 60304
708-383-2257

UIC/Chicago

1740 W. Taylor St.
Room C100
Chicago, IL 60612
312-996-6450

Schaumburg

1701 E. Woodfield Rd.,
Ste 555
Schaumburg, IL 60173
847-619-1701

Hickory Hills

8641 W. 95th St.
Hickory Hills, IL 60457
708-599-8336

Schererville

6629 W. Lincoln Hwy.
(Rt. 30), Suite 1
PO Box 507
Schererville, IN 46375
219-864-9501
