

Hip Disarticulation (HD) and Hemi Pelvectomy (HP) Prostheses

General Description:

According to the 2000 census, the population of the United States is 281 million people. Of this number, 1.5 million are amputees. Less than 1% of these individuals or approximately 10,000 people are either Hip Disarticulation (HD) or Hemi Pelvectomy (HP) amputees. It is estimated that energy consumption required to walk at this level is 200% of normal walking. As a result, many individuals choose not to be full-time prosthetic users. Due to the fact that HD/HP's comprise a very small segment of the amputee population and of this population many are not full time users, there have not been many advances in fitting techniques at this level. The "Canadian Design" for fitting, introduced in 1954, is still the standard for fitting in the United States today. Early edition HD/HP sockets were fabricated of leather with a metal frame. Over time, the leather shapes itself to the residual limb resulting in a more comfortable fit. The main drawback of leather is that it deteriorates quickly from perspiration. Plastic laminated sockets, used in the fifties, remedied the problem of durability. The challenge has been to create a socket that is strong but also flexible enough to put on easily.

There have been great advances in materials and components in prosthetics in the last 10-15 years. The industry has begun to use thermoplastic materials such as polypropylene and copolymer to fabricate sockets and have achieved good results (See figure1). Flexible plastics are used as the interface between the individual's limb and the prosthesis. A Prosthetist can choose from many different types of flexible plastics.



Arlington Heights
617 E. Golf Road, Suite 108
Arlington Heights, IL 60005
847-437-3929

Bannockburn
2101 Waukegan Road,
Suite 104
Bannockburn, IL 60015
847-444-0690

Lincoln Park/Chicago
2551 N. Clark St., Suite 200
Chicago, IL 60614
773-472-3663

Rush/Chicago
1725 W. Harrison St.
Suite 220
Chicago, IL 60612
312-942-2011

UIC/Chicago
1740 W. Taylor St.
Room C100
Chicago, IL 60612
312-996-6450

Hickory Hills
8641 W. 95th St.
Hickory Hills, IL 60457
708-599-8336



Naperville
1551 Bond St., Suite 111
Naperville, IL 60563
630-637-4638

Oakbrook Terrace
1S376 Summit Ave., Ct. E
Oakbrook Terrace, IL 60181
630-424-0392

Oak Park
401 Harrison Street
Oak Park, IL 60304
708-383-2257

Schererville
6629 W. Lincoln Hwy.
(Rt. 30), Suite 1
PO Box 507
Schererville, IN 46375
219-864-9501

Utilizing these materials, the prosthetist has the option of making the socket more or less flexible depending on the patient's needs (See figure 2). With HP sockets, the plastic of the socket must extend higher on the torso due to the lack of bony anatomy with which to bear weight. With flexible plastics, the plastic remains high on the torso but is more flexible and thus far more comfortable to the individual. Body heat helps make the plastics more flexible and, like the leather sockets, reshape itself to the user's needs (See figure 3). This material is strong, but due to its flexibility, it cannot support a person's body weight by itself on a prosthetic hip joint without collapsing. Similar to the leather and metal frame technique, a rigid frame is fabricated over the flexible socket. Materials such as carbon graphite and polypropylene are used to form the main support and structure of the socket. The rigid frame encompasses the prosthetic hip joint and the socket.

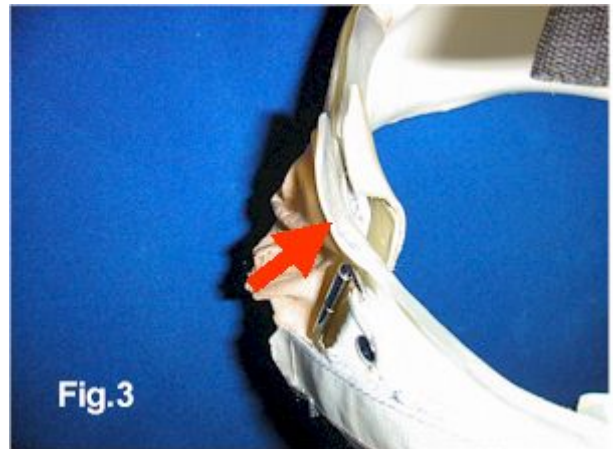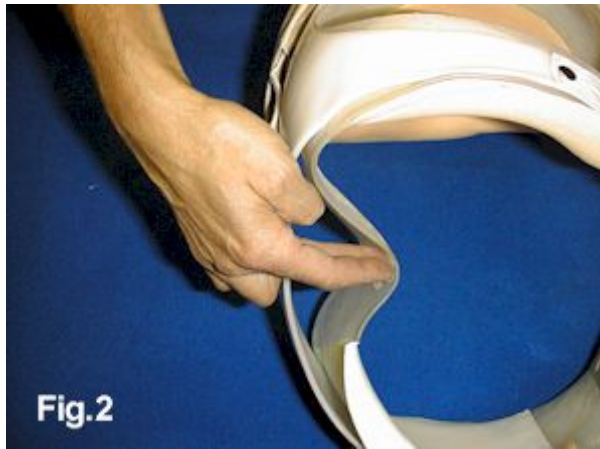

Arlington Heights
617 E. Golf Road, Suite 108
Arlington Heights, IL 60005
847-437-3929

Bannockburn
2101 Waukegan Road,
Suite 104
Bannockburn, IL 60015
847-444-0690

Lincoln Park/Chicago
2551 N. Clark St., Suite 200
Chicago, IL 60614
773-472-3663

Rush/Chicago
1725 W. Harrison St.
Suite 220
Chicago, IL 60612
312-942-2011

UIC/Chicago
1740 W. Taylor St.
Room C100
Chicago, IL 60612
312-996-6450

Hickory Hills
8641 W. 95th St.
Hickory Hills, IL 60457
708-599-8336

Naperville
1551 Bond St., Suite 111
Naperville, IL 60563
630-637-4638

Oakbrook Terrace
1S376 Summit Ave., Ct. E
Oakbrook Terrace, IL 60181
630-424-0392

Oak Park
401 Harrison Street
Oak Park, IL 60304
708-383-2257

Schererville
6629 W. Lincoln Hwy.
(Rt. 30), Suite 1
PO Box 507
Schererville, IN 46375
219-864-9501

The height of the frame is usually lower than the flexible interface to achieve the benefit of the flexible plastic. Depending on the condition of the residual limb, areas of the frame can be cut out to help relieve problem areas in the socket (See figure 4).



Pieces of silicone gel have been used on the bottom of hip disarticulation sockets to achieve a more comfortable fit (See figure 5). Silicone gel acts as a shock absorber while at the same time works to decrease shear forces on the residual limb. Inevitably, there is movement within the socket. With the addition of the silicone gel on the bottom, prosthetists are able to reduce discomfort and abrasions from the sheering movement. The silicone gel can also be incorporated anywhere on the socket to protect bony prominences. It should be noted that this technique works best when incorporated into the initial fabrication of the socket. Retrofitting the socket at an area of increased pressure by



Arlington Heights
617 E. Golf Road, Suite 108
Arlington Heights, IL 60005
847-437-3929

Bannockburn
2101 Waukegan Road,
Suite 104
Bannockburn, IL 60015
847-444-0690

Lincoln Park/Chicago
2551 N. Clark St., Suite 200
Chicago, IL 60614
773-472-3663

Rush/Chicago
1725 W. Harrison St.
Suite 220
Chicago, IL 60612
312-942-2011

UIC/Chicago
1740 W. Taylor St.
Room C100
Chicago, IL 60612
312-996-6450

Hickory Hills
8641 W. 95th St.
Hickory Hills, IL 60457
708-599-8336

Naperville
1551 Bond St., Suite 111
Naperville, IL 60563
630-637-4638


Oakbrook Terrace
1S376 Summit Ave., Ct. E
Oakbrook Terrace, IL 60181
630-424-0392

Oak Park
401 Harrison Street
Oak Park, IL 60304
708-383-2257

Schererville
6629 W. Lincoln Hwy.
(Rt. 30), Suite 1
PO Box 507
Schererville, IN 46375
219-864-9501

adding silicone gel will only increase the pressure at that site causing further irritation and skin breakdown.

The biggest advance in the fitting of HD prostheses has been in designing sockets that capture as much of the underlying skeletal anatomy as possible. Careful casting and a good knowledge of anatomy helps the prosthetist to utilize these contours to improve weight bearing surfaces as well as control movement within the socket. Much attention is now paid to containing the ischium (sitting bone) in the socket (See figure 6) . This may help the individual to achieve a more natural gait but also may reduce energy consumption. By containing the ischium within a HD prosthesis, excessive movement of the pelvis is minimized, thereby giving the individual more control of the socket. Any extra movement of the pelvis can result in the possibility of abrasions or breakdown, and may result in the individual requiring more energy to walk.

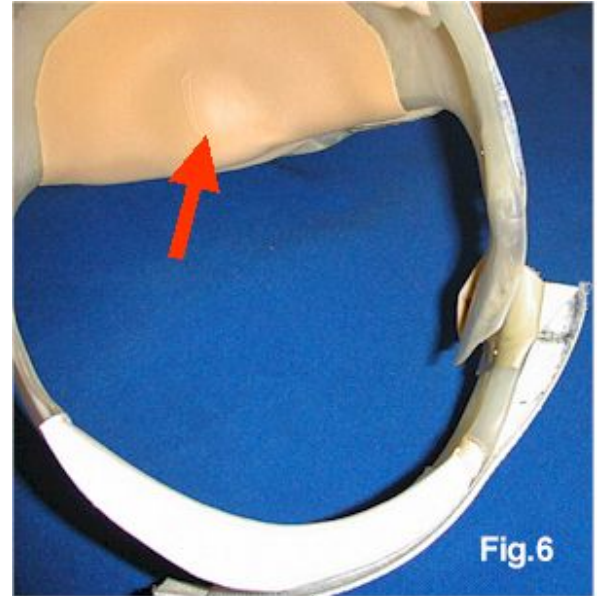

Proper selection of components plays a crucial part in a successful fitting. Many new components in the prosthetic industry have been introduced over the past few years. Computerized knees and polycentric knees with stance flexion features are two designs that make ambulation easier, safer, and more natural. Torsion adaptors reduce shear forces inside the socket while thigh turntables allow individuals to sit more comfortably. Hip Flexion bias systems assist in getting the thigh section out to take the next step more quickly, thus reducing energy consumption. There are many different types of prosthetic feet available today. Lightweight componentry such as aluminum, titanium, and carbon are all used to reduce the overall weight of the prosthesis. With all of the different components available, the user and his or her prosthetist can find the one that best suits the user’s needs.

<p>Arlington Heights 617 E. Golf Road, Suite 108 Arlington Heights, IL 60005 847-437-3929</p>	<p>Bannockburn 2101 Waukegan Road, Suite 104 Bannockburn, IL 60015 847-444-0690</p>	<p>Lincoln Park/Chicago 2551 N. Clark St., Suite 200 Chicago, IL 60614 773-472-3663</p>	<p>Rush/Chicago 1725 W. Harrison St. Suite 220 Chicago, IL 60612 312-942-2011</p>	<p>UIC/Chicago 1740 W. Taylor St. Room C100 Chicago, IL 60612 312-996-6450</p>
<p>Hickory Hills 8641 W. 95th St. Hickory Hills, IL 60457 708-599-8336</p>	<p>Naperville 1551 Bond St., Suite 111 Naperville, IL 60563 630-637-4638</p>	<p>Oakbrook Terrace 1S376 Summit Ave., Ct. E Oakbrook Terrace, IL 60181 630-424-0392</p>	<p>Oak Park 401 Harrison Street Oak Park, IL 60304 708-383-2257</p>	<p>Schererville 6629 W. Lincoln Hwy. (Rt. 30), Suite 1 PO Box 507 Schererville, IN 46375 219-864-9501</p>

Application:

1. First put on a donning sock. This may be worn with or without underwear. Some amputees prefer not to wear a sock. Some amputees have specially made underwear that has the amputated side sewed shut.
2. While standing, pull the flaps of the socket open and slide the amputated side into the prosthetic socket.
3. Tighten the straps firmly across the front of the body, closing in the flaps of the socket. The socket may be secured with Velcro straps or buckles. Make sure the wedges on the sides of the socket fit firmly above the hipbones.

Cleaning and Maintenance:

The inside of the socket can be cleaned with warm, soapy water or anti-bacterial baby wipes. Do not submerge the socket in water and keep the prosthesis away from heat, as this can damage the socket.

Tips and Problem Solving:

Most sores or abrasions that occur are caused by movement within the socket, usually due to the socket being loose. Make sure the straps are tight across the front of the socket. If you have recently lost weight, you may need to see your prosthetist to have the socket padded or lined.

If you have recently gained weight, sores may also develop. See your prosthetist to see if adjustments can be made to accommodate the weight change. Otherwise, you may need a new socket.

Depending on the type of hip joint, knee, and foot, scheduled maintenance checks may be needed with the prosthetist. Please consult your prosthetist as to your schedule. Between checkups, let your prosthetist know if you experience any problems such as a squeaky hip joint, changes in knee function, or alignment issues.

Click here for information on [Knees](#).

Click here for information on [Feet](#).

<p>Arlington Heights 617 E. Golf Road, Suite 108 Arlington Heights, IL 60005 847-437-3929</p>	<p>Bannockburn 2101 Waukegan Road, Suite 104 Bannockburn, IL 60015 847-444-0690</p>	<p>Lincoln Park/Chicago 2551 N. Clark St., Suite 200 Chicago, IL 60614 773-472-3663</p>	<p>Rush/Chicago 1725 W. Harrison St. Suite 220 Chicago, IL 60612 312-942-2011</p>	<p>UIC/Chicago 1740 W. Taylor St. Room C100 Chicago, IL 60612 312-996-6450</p>
<p>Hickory Hills 8641 W. 95th St. Hickory Hills, IL 60457 708-599-8336</p>	<p>Naperville 1551 Bond St., Suite 111 Naperville, IL 60563 630-637-4638</p>	<p>Oakbrook Terrace 1S376 Summit Ave., Ct. E Oakbrook Terrace, IL 60181 630-424-0392</p>	<p>Oak Park 401 Harrison Street Oak Park, IL 60304 708-383-2257</p>	<p>Schererville 6629 W. Lincoln Hwy. (Rt. 30), Suite 1 PO Box 507 Schererville, IN 46375 219-864-9501</p>
